

TECO® *Fast Cryptococcus neoformans* Antigen Lateral Flow Assay

Detection of the cryptococcal capsular polysaccharide antigen (Ag) in human serum and cerebral spinal fluid (CSF) as a diagnostic reference of cryptococcal meningitis infection

Cryptococcus neoformans is a yeast that is ubiquitous in the environment. Infections are life-threatening in immunosuppressed patients; therefore, a rapid and precise diagnosis is imperative. The LFA offers a fast and simple way to test at-risk patients and those exhibiting symptoms.




- Fluorescence-based lateral flow assay designed for objective readout via analyzer
- Clinically validated for CSF and serum samples
- Simple and fast: no sample treatment required, results after 20 minutes
- Qualitative results: evaluation based on clinically validated cut-off value
- Easy data management: transmission of results to internal printer, LIS system or as Excel file



TECO® Fast Cryptococcus neoformans Antigen Lateral Flow Assay

Human serum and cerebral spinal fluid can be tested for cryptococcal capsular polysaccharide glucuronoxylomannan using double antibody sandwich fluorescence immunochromatography.

Test procedure

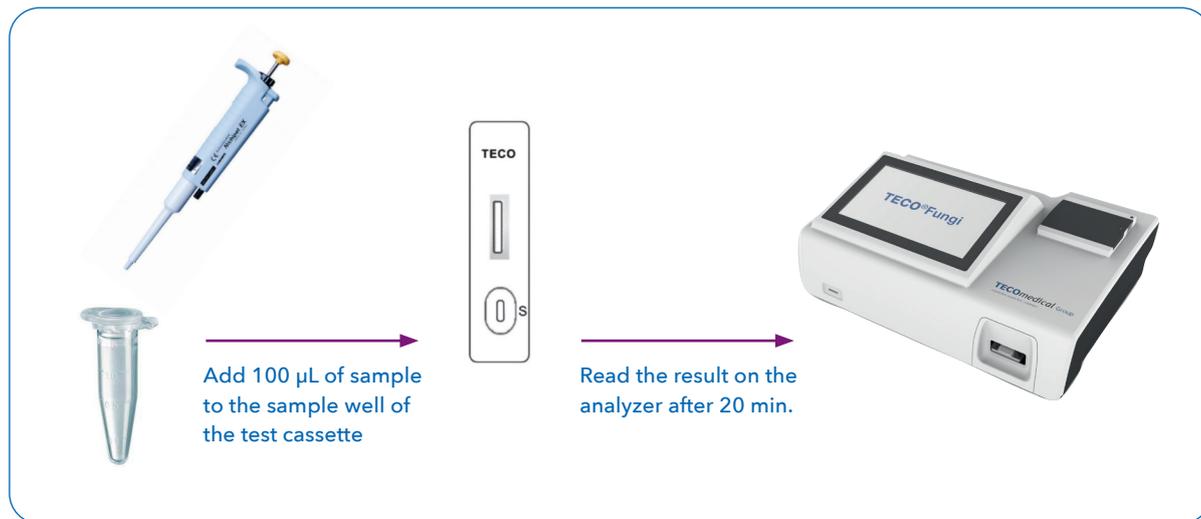

Performance data

- Sensitivity: 100% (CSF), 97% (serum)
- Specificity: 100% (CSF), 98.1% (serum)
- Cut-off: 0.5 ng/mL
- Limit of detection: 0.1 ng/mL

 Please consult instructions for use for full information

For professional use only. Not intended for self-testing or near-patient testing.

Product name: TECO® Fast Cryptococcus neoformans Antigen Lateral Flow Assay

Catalogue number: TE1085

Kit content: 50 cassettes + controls

UDI-DI: 7640146270184

For further information please contact:

TECOmedical Group

 **TECOmedical AG**
 Gewerbstrasse 10
 4450 Sissach
 Switzerland
 Phone: +41 61 985 81 00
 Fax: +41 61 985 81 09
 Mail: info@tecomedical.com

Germany
TECOmedical GmbH
 Hotline: 0800 985 99 99

Austria
TECOmedical AG
 Hotline: 0800 20 40 66

 **Eurobio Scientific SA**
 7, avenue de Scandinavie
 ZA de Courtaboeuf
 91940 Les Ulis, France
 Phone: +33 1 69 79 64 80
reglementaire@eurobio-scientific.com

