

# TECO<sup>®</sup> *Fast Aspergillus Galactomannan Antigen Lateral Flow Assay*

Detection of *Aspergillus galactomannan antigen (Ag)* in serum and bronchoalveolar lavage fluid (BAL) as a diagnostic reference of *Aspergillus* infection

Without timely treatment, the mortality rate of invasive aspergillosis ranges from 60 to 100%; thus, prompt determination of galactomannan antigen may be essential.



- Fluorescence-based lateral flow assay designed for objective readout via analyzer
- Validated for serum and BAL samples
- Simple and fast: < 40 min. from native sample to result
- Qualitative results: evaluation based on clinically validated cut-off index value
- Easy data management: transmission of results to internal printer, LIS system or as Excel file



## TECO® Fast Aspergillus Galactomannan Antigen Lateral Flow Assay

The galactomannan antigen is detected using double antibody sandwich fluorescence immunochemistry with monoclonal anti-galactomannan antibodies.

Comparison data between a Galactomannan ELISA assay and the Aspergillus Galactomannan Ag LFA:

	Reference method result		Total
	Positive	Negative	
Result: SERUM			
Positive	97	4	101
Negative	10	156	166
Total	107	160	267

	Reference method result		Total
	Positive	Negative	
Result: BAL			
Positive	73	5	78
Negative	6	95	101
Total	79	100	179



### Performance data

- Positive coincidence rate: 90.7% (serum); 92.4% (BAL)
- Negative coincidence rate: 97.5% (serum); 95.5% (BAL)
- Cut-off index 0.5: negative < 0.5 ≥ positive
- Limit of detection: 1 ng/mL (serum and BAL)

 Please consult instructions for use for full information

For professional use only. Not intended for self-testing or near-patient testing.

Product name: TECO® Fast Aspergillus Galactomannan Antigen Lateral Flow Assay

Catalogue number: TE1069

Kit content: 50 cassettes + controls

UDI-DI: 7640146270108

For further information please contact:

## TECOmedical Group

 **TECOmedical AG**  
 Gewerbstrasse 10  
 4450 Sissach  
 Switzerland  
 Phone: +41 61 985 81 00  
 Fax: +41 61 985 81 09  
 Mail: info@tecomedical.com

**Germany**  
**TECOmedical GmbH**  
 Hotline: 0800 985 99 99

**Austria**  
**TECOmedical AG**  
 Hotline: 0800 20 40 66

 **Eurobio Scientific SA**  
 7, avenue de Scandinavie  
 ZA de Courtaboeuf  
 91940 Les Ulis, France  
 Phone: +33 1 69 79 64 80  
 reglementaire@eurobio-scientific.com

