

## Information

*always your partner*

## TECO<sup>®</sup>Fast Aspergillus IgG



## Antibody Lateral Flow Assay

For rapid diagnosis of chronic cavitary pulmonary aspergillosis (CCPA) and allergic bronchopulmonary aspergillosis (ABPA)

- Fluorescence-based lateral flow assay designed for readout via reader
- Semiquantitative assay; calculation of concentration value in AU/mL
- Validated for serum samples
- Time to result: 15 minutes after diluting the sample
- Clinical sensitivity: 88.9% (n = 143)
- Clinical specificity: 91.3% (n = 143)
- Limit of detection: 14 AU/mL, cut-off value of 135 AU/mL




## TECO® Fast Aspergillus IgG Antibody LFA – Sample Preparation




Mix serum sample 1:200 with Dilution Buffer and vortex for 10 seconds

## TECO® Fast Aspergillus IgG Antibody LFA – Assay Procedure

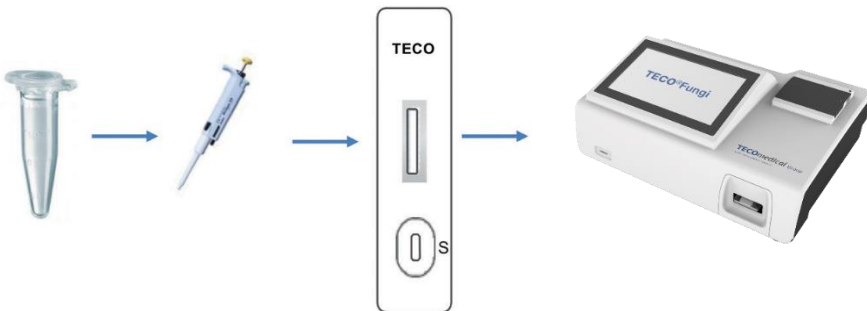

Add 100 µL of diluted serum to the sample well of the test cassette

Insert the test cassette into the analyzer

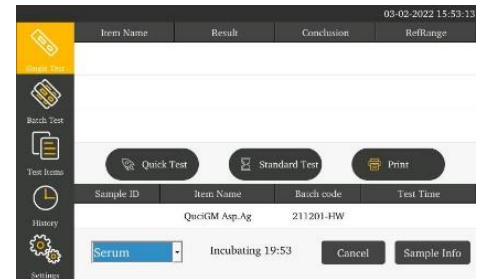

Result available in 15 min via:

- printer
- USB (.xls file)
- LIS
- internal memory

## TECO® Fast Aspergillus IgG Antibody Assay Catalogue No.: TE 1070

Für weitere Informationen wenden Sie sich bitte an:  
For further information please contact:  
Pour plus d'information, veuillez contacter:

[www.tecomedical.com](http://www.tecomedical.com)



A EUROBIO SCIENTIFIC COMPANY

**Switzerland / Headquarters**  
**TECO medical AG**  
Gewerbstrasse 10  
4450 Sissach  
Phone +41 61 985 81 00  
Fax +41 61 985 81 09  
Mail [info@tecomedical.com](mailto:info@tecomedical.com)

**Germany**  
**TECO medical GmbH**  
Wasserbreite 57  
32257 Bünde  
Phone +49 52 23 985 99 99  
Fax +49 52 23 985 99 98  
Mail [info@tecomedical.com](mailto:info@tecomedical.com)

**Benelux**  
**TECO medical Benelux BV**  
Prins Willem-Alexanderlaan 301  
7311 SW Apeldoorn, The Netherlands  
Phone +31 30 307 87 30  
Fax +31 30 307 49 39  
Mail [benelux@tecomedical.com](mailto:benelux@tecomedical.com)

**Austria**  
**TECO medical AG**  
Phone 0800 20 40 66  
Fax 0800 20 40 55  
Mail [info@tecomedical.com](mailto:info@tecomedical.com)